Delta Systems

Table of Contents

EN.....	2
CS.....	4
DA.....	6
DE.....	8
ES.....	9
ET.....	12
FI.....	14
FR.....	16
HU.....	17
IT.....	20
NL.....	22
NO.....	24
PL.....	26
PT.....	27
SK.....	30
SL.....	32
SV.....	34

EN

Instructions for Use

IMPORTANT

Read Instructions carefully before use

A SPECIFICALLY FORMULATED CARE REGIME FOR GAS PERMEABLE CONTACT LENSES ONLY

Stage 1 - cleaning

Wash, rinse and dry your hands thoroughly before handling your lenses.

Place each lens in the palm of your hand and apply 2-3 drops of Delta Cleaner.

Gently rub both sides of the lens with tip of forefinger for 15-20 seconds.

Stage 2 - rinsing

Rinse cleaned lenses with Delta Plus DSW solution or sterile saline.

CARE OF LENS CASE

After using your contact lens case, thoroughly rinse with Delta Plus DSW solution or sterile saline. Once a week thoroughly clean your soaking case paying particular attention to the compartments and caps. **DO NOT USE SOAP OR DETERGENTS** and allow to air dry. Contact lens cases should be kept dry when not in use and renewed frequently as recommended by your contact lens practitioner.

Stage 3 - disinfecting

Remove the lid from your contact lens case and place your left lens in the left compartment and right lens in the right compartment of your case, fill the case almost to the brim with Delta Plus DSW and securely tighten the cap. Leave lenses to soak for at least four hours or overnight. It is recommended that lenses are re-disinfected after prolonged periods of storage i.e., seven days or more.

Stage 4 - rinsing

After disinfection, rinse your lenses with Delta Plus DSW solution before insertion into the eye. Discard any solution remaining in your lens case and allow to air dry.

Lens Comfort

To improve comfort and wettability a few drops of Delta DSW solution may be applied to both surfaces of the lens prior to insertion into the eye.

Protein Removal

The weekly use of protein remover tablets may be necessary for some patients. Please consult your contact lens practitioner.

PRECAUTIONS

DO NOT MIX WITH OTHER FLUIDS EXCEPT AS DIRECTED. Always use the complete Delta aftercare system for Gas Permeable lenses. Do not substitute with products from other systems. Do not introduce Delta cleaner directly into the eye. Consult your contact lens practitioner before using any eye medication while wearing your contact lenses. In the event of persistent eye irritation, discontinue use, remove lens from the eye and consult your contact lens practitioner. Discard Delta Plus DSW and Delta Cleaner 3 months after first opening. Keep

all contact lens care products away from children. Do not remove cap from case during contact lens storage. Do not allow the dropper tip to touch any surface. Always replace bottle cap after use. Store between 8°C and 30°C. Do not refrigerate Delta Cleaner. Do not rinse contact lenses or lens case with water directly from the tap. Do not use if tamper-evident seal is damaged.


Ingredients

Delta Cleaner:

Disodium Cocoamphodiacetate, water

Delta Plus Disinfecting, Soaking & Wetting Solution

Polyhexanide 0.0001% w/v, EDTA, sodium phosphates, poloxamer, sodium chloride, water

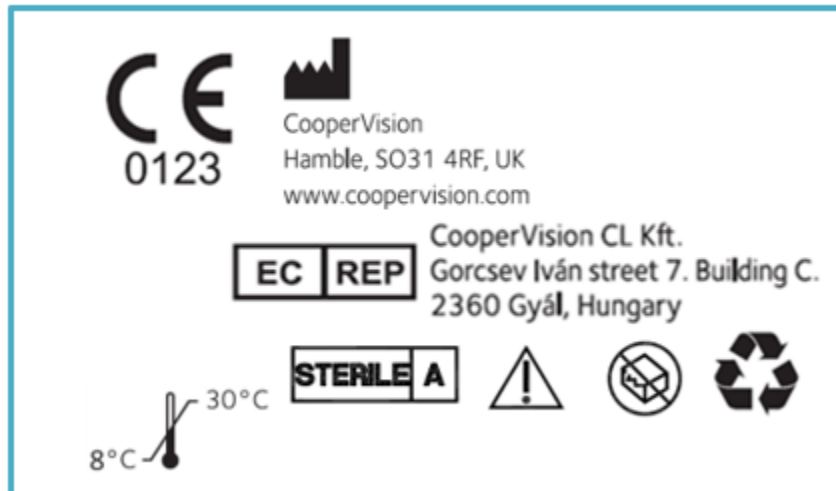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