## Preservative Free Saline

### Table of Contents

EN.....	2
BG.....	3
CS.....	4
DA.....	5
DE.....	6
ES.....	7
ET.....	8
FI.....	9
FR.....	10
EL.....	11
HU.....	12
IT.....	13
NL.....	14
NO.....	15
PL.....	16
PT.....	17
RU.....	18
SK.....	19
SL.....	20
SV.....	21
TR.....	22

INGREDIENTS / SLOŽENÍ / СЪСТАВКИ / INGREDIENSER / BESTANDDELEN / KOOSTIS / AINESOSAT /  
COMPOSANTS / INHALTSSTOFFE / ΣΥΣΤΑΤΙΚΑ / ÖSSZETEVŐK / INGREDIENTI / SKŁAD /  
INGREDIENSER / INGREDIENTES / СОСТАВ / ZLOŽENIE / SESTAVINE / INGREDIENSER / İÇERİK:  
0.005%w/v hydrogen peroxide, sodium chloride, sodium phosphates

## EN

### Instructions for Use

#### IMPORTANT

read instruction leaflet carefully before use

#### PRESERVATIVE FREE

#### SALINE SOLUTION

#### SUITABLE FOR RINSING ALL CONTACT LENSES

**IMPORTANT:** Do not change from the solutions recommended by your contact lens practitioner. Preservative Free Saline is not for use as a disinfectant.

#### DESCRIPTION:

Preservative Free Saline is an isotonic, sterile solution, buffered to pH7.

#### USES:

- For the rehydration of all soft contact lenses prior to disinfection.
- As a final rinse for all contact lenses before re-insertion into the eyes.
- For dissolving tablets where saline buffered to a pH of 7 is recommended.
- For rinsing contact lens storage cases.

#### DIRECTIONS FOR USE:

Wash, rinse and dry your hands thoroughly. Unscrew cap and pour required amount of Preservative Free Saline solution. Solution is ready to use.

#### DISSOLVING TABLETS

Preservative Free Saline may be used to dissolve protein remover tablets where saline buffered to a pH of 7 is recommended.

Follow the directions for use on the tablet pack

#### PRECAUTIONS:

**DO NOT MIX WITH OTHER FLUIDS EXCEPT AS DIRECTED.**

**DO NOT USE PRESERVATIVE FREE SALINE TO CLEAN AND DISINFECT YOUR LENSES.**

Do not use if tamper-evident seal is broken. Consult your contact lens practitioner before using any eye medication while wearing your contact lenses. In the event of persistent eye irritation discontinue use, remove lens from eye and consult your contact lens practitioner. Do not swallow the solution.

Keep all contact lens care products away from children. Do not remove cap from case during contact lens storage. Always replace bottle cap after use. Once bottle is open, discard any remaining solution after 3 months. Store between 4°C to 30°C. Do not rinse lenses or lens cases with water directly from the tap

Use before expiration date printed on the carton and bottle.

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