## Comfort Drops

### Table of Contents

EN.....	1
CS.....	3
DA.....	4
DE.....	5
ES.....	6
FI.....	7
FR.....	8
HU.....	9
IT.....	10
NL.....	11
NO.....	12
PL.....	13
PT.....	14
RO.....	15
SK.....	16
SL.....	17
SV.....	18

CooperVision  
Hamble, SO31 4RF, UK  
[www.coopervision.com](http://www.coopervision.com)

CooperVision CL Kft.  
Gorcsev Iván street 7. Building C.  
2360 Gyál, Hungary

## EN

### Instructions for Use

#### IMPORTANT

Read Instructions carefully before use

#### Indications for Use:

Suitable for use with contact lenses. Comfort Drops have lubricating and moisturising properties, helping to relieve eye discomfort and irritation caused by dust, pollen and air pollution.

Intended users: Users that need help to relieve eye discomfort and irritation caused by dust, pollen, and air pollution. Suitable for contact lens wearers.

#### INSTRUCTIONS FOR USE:

Wash, rinse and dry your hands thoroughly. Unscrew cap. Tilt head backwards. Place one or two drops into each affected eye when required (once or twice daily).

#### PRECAUTIONS:

- To avoid contamination do not touch dropper tip on any surface and avoid direct contact with the eye.
- Do not swallow the solution.
- Discard 3 months after opening.
- Keep out of reach of children.
- Store between 4-25°C.
- If you experience prolonged discomfort or any problems please contact your eyecare practitioner.
- Keep the bottle tightly closed when not in use.
- Use before expiration date printed on the carton and bottle.
-  Do not use if tamper-evident seal is broken.

#### Contra-indications:

Do not use if allergic to any ingredient.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and applicable competent authority.

The sterile barrier for solution is the unopened bottle cap.

Disposal: use standard waste, recycle whenever possible.

INGREDIENTS: Polyhexanide 0.0001%w/v, HPMC, EDTA, sodium phosphates, poloxamer, sodium chloride, water.

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